



Patient Transfer



PATIENT TRANSFER RANGE



GlobalMedics
Medical and Hygiene Supply Group



THE COMPANY

The Global Medics Group is a young dynamic company servicing the Australasian market with unique and niche products to the four Key market segments -

AGED CARE



THEATRE SURGICAL



HOSPITAL



PHARMACY/IT



The Company's vision is to improve quality of Life for all patients and their healthcare providers by focusing on key specialities - **Infection Control to reduce Hospital acquired infections / Pharmacy Automation and Management / Contingence Management-Aged Care / Lean Inventory Management / Mobile Computing** all to improve the patient outcomes. Forging “**Partners for Life**” through understanding your needs and ensuring our product and technology are there when the care is needed.



OUR MISSION

To forge “Partners for Life” by delivering innovative products, simple and effective solutions to improve quality of life for both patients and healthcare providers.

FOR ALL ENQUIRIES PLEASE CONTACT OUR HELPLINE

AUSTRALIA

customerservice@globalmedicsgroup.com

www.globalmedicsgroup.com

+1300 197 261

NEW ZEALAND

customerservice@globalmedics.co.nz

www.globalmedics.co.nz

0800 GLOMED 45 66 33



GlobalMedics
Medical and Hygiene Supply Group



CONTENTS

Overview	1
PATIENT TRANSFER Contents	2
FLEXISLIDE Single Use Patient Transfer Slide Sheet	3-4
SALLYBOARD Patient Transfer Board	5
SLIPPERY SALLY Single Patient Use Slide Sheet	6
SLIPPERY SALLY Re-usable Fully Hemmed Slide Sheet	7
SLIPPERY SALLY Single Use Patient Transfer Tubes	8
SLIPPERY SALLY Re-usable Transfer Tubes	9
Order Form	10




www.globalmedicsgroup.com
www.globalmedics.co.nz

FLEXISLIDE™ is a single use, lateral transfer slide sheet, designed to make patient handling easy and safe, by helping to prevent back strain and injury and assisting in the prevention of cross infection. It is ideal for use in departments across the hospital when moving a patient between two surfaces in a pre/peri-operative OR ER environment. The product is lightweight and inexpensive but high quality and immensely strong - the material has been tested to over 400kg, wet or dry.



- **COMPACT AND EASY TO STORE**
- **'EASY-SLIDE' DESIGN** sufficiently stable to remain stationary on a tilted operating table, but slides easily to move the patient in a controlled manner
- **LATEX FREE**
- **WATER RESISTANT**
- **REMAINS UNDER THE PATIENT** so is always ready for use
- **BLUE COLOUR** ensures quick, easy identification
- **REPLACES UNNECESSARY SHEETS** etc thereby reducing laundry costs and wastage
- **BARRIER AGAINST BACTERIA**
- **DEDICATED HAND HOLES** ensuring secure grip and reduced wrist stress (refer below)
- **INCREASED SAFETY** for both patients and staff

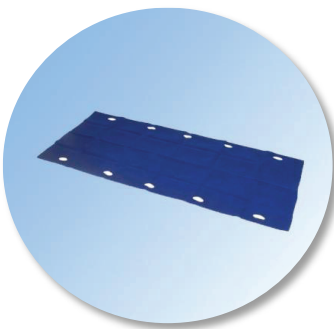

Advantages of dedicated hand hole for gripping

FLEXISLIDE™ also features ten dedicated hand holes for gripping that minimise awkward twisting of the wrist and ensure a secure hand hold. These handles reduce the risk of injury and wrist stress by enabling the user to hold the slide with upturned palms, using the stronger muscles of the arms.

Gripping with the palms down uses weaker muscles in the arms and results in elevation of the shoulder girdle. This puts the shoulder, elbow, and wrists joints in awkward postures under load, which increases the risk of injury.

FLEXISLIDE

FOR THEATRE PATIENT TRANSFER



Instructions for use

Always risk-assess each patient manoeuvre following the guidelines in your hospital 'Movement and Handling Policy'. Take into account factors such as patient size and weight. Always ensure that the correct numbers of staff are available before making any transfer.

1. Ensure that all trolley/bed brakes are engaged.
2. Always use a bridging device between the 2 surfaces when transferring patients (e.g. Pat Slide or Easy slide).
3. Place Flexislide™ on the trolley, ensuring that the shiny / hemmed side is facing the trolley surface.
4. Always place the patient's pillow on top of the Flexislide™ and under the patient's head.
5. Ensure the Flexislide™ is as flat as possible underneath the patient's head and feet.
6. To transfer the patient use the hand hole most comfortable for your own height and size.
7. When transferring patients always use a combined push and pull technique as recommended in your hospital 'Movement and Handling Policy'.
8. Flexislide™ is capable of withstanding the lateral transfer of patients weighing up to 424KG when using a combined push and pull technique.
9. To remove or place a new Flexislide™ under an immobile patient do so as if changing a bed sheet. Flexislide™ is folded to assist you in this process.
10. Once the patient has been returned to their bed remove the Flexislide™.



Important Notes

- NEVER LIFT a patient using Flexislide™
- When transferring patients DO NOT use extension handles or straps of any type
- It's important to ensure that fluids are not left to pool on the surface of the Flexislide™ or beneath the patient (use of absorbent pads may help here)
- If the Flexislide™ has remained clean it can be kept with the patient and can continue to be used for additional lateral transfers while that specific patient remains in hospital

CODE	DESCRIPTION	CTN QTY	SIZE
MW1061	FLEXSLIDE Extra Wide Transfer Slide Sheet	25	115x196cm
MW1029	FLEXISLIDE Standard	50	89x196cm



SALLYBOARD

PATIENT TRANSFER BOARDS

WITH 4 GRIP STRIPS ON UNDERSIDE

LIGHTWEIGHT FOR EASY PATIENT TRANSFER

Why use the Sallyboard Range of Patient Transfer Boards?

- CONSTRUCTED FROM LIGHTWEIGHT, HDPE much lighter than many transfer boards on the market
- HANGING HOLE UP ONE END
- S.W.L. 300KG (NATA APPROVED LABORATORY) for Medium, Large and Theatre SallyBoards
- TGA REGISTERED
- MEDIUM, LARGE AND THEATRE SALLYBOARDS all have 3 handles on each side for ease of manoeuvring
- SMOOTH TOP SURFACE AND CURVED EDGES FOR EASY PATIENT TRANSFER
- 4 GRIP STRIPS ON UNDERSIDE TO KEEP BOARD IN PLACE
- GOOD LEVEL OF FLEXIBILITY IN THE BOARD where bed / barouche heights are not always at the same level
- RADIOLUCENT can be used for X-rays



CODE	DESCRIPTION	CTN QTY	SIZE
MT SBL	SALLYBOARD LARGE Patient Transfer Board	1	180cm x 58cm x 5mm



SLIPPERY SALLY

SINGLE PATIENT USE SLIDE SHEET

SlipperySally®

SLIPPERY SALLY® SINGLE PATIENT USE SLIDE SHEETS can greatly improve infection control as well as improve manual handling over reusable slide sheets.


During general ward use, Slippery Sally Single Patient Use Slide Sheets can be used for several months and simply wiped down as needed or, in high infections, may be disposed of after use.

Slippery Sally® Single Patient Use Slide Sheets give you:

- REDUCED SPREAD OF INFECTION WHICH MAY OCCUR THROUGH REUSE OF SLIDE SHEETS
- SIGNIFICANT COST SAVINGS
- NO LAUNDERING COST
- ABILITY TO QUICKLY GRAB ONE IN AN EMERGENCY AND USE ON EITHER SIDE very slippery on both sides
- REDUCED CHANCE OF INFECTION I90T polyester taffeta - S.W.L. 360kg
- EASY IDENTIFICATION bright yellow
- IMPROVED MANUAL HANDLING no excuse for staff not to use a slide sheet because they cannot find it or it has not returned from the laundry

Slippery Sally® Single Patient Use Slide Sheets are used by:

- THEATRE / EMERGENCY DEPARTMENTS / AMBULANCE SERVICES / MRSA PATIENTS
- HOME CARE SERVICES - Ideal for short term clients
- HOSPITALS / AGED CARE WHERE EXPENSIVE SLIDE SHEETS ARE LOST THROUGH THEFT OR LINEN SERVICING

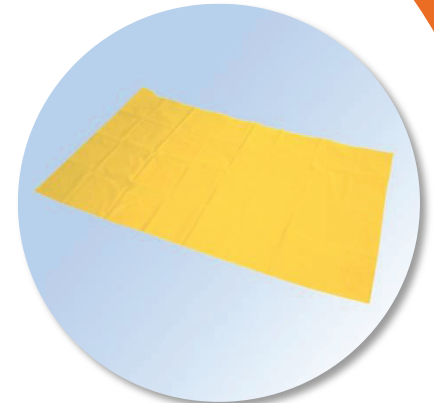

CODE	DESCRIPTION	CTN QTY	SIZE
MT SLIP1-D50	SLIPPERY SALLY Single Patient Use Slide Sheet	50	1.0m x 1.45m
MT SLIP2-D50	SLIPPERY SALLY Single Patient Use Slide Sheet	50	2.0m x 1.45m



NEW!
Premium
Slip Fabric


The PREMIUM SLIPPERY SALLY® RE-USABLE range of machine washable slide sheets has a slip coating on both sides and a quiet soft feel. This Premium Slippery Sally® slips extremely well and consistently on both sides and in all directions. It is the fabric of choice for some of Australia's leading manual handlers.

Premium Slippery Sally® Re-Usable Slide Sheets are Available in Two Options:

- **HEAT SEALED EDGES** reduces the chance of infection (from stored pathogens)
- **FULLY-HEMMED** with a 12 x 8cm identification label sewn on is great for reducing Slippery Sally loss at the laundry as well as allocating Slippery Sally's by ward or patient

Premium Slippery Sally® Re-Usable Slide Sheets:

- **SILICON COATED BOTH SIDES** easy to use and very slippery
- **SLIPS CONSISTENTLY IN ALL DIRECTIONS / BOTH SIDES** no right way up
- **MADE OF SOFT POLYESTER TAFFETA**
- **RETAIN THEIR SLIP FOR OVER 150 WASHES**
- **MACHINE WASHABLE UP TO 75 DEGREES**
- **TUMBLE DRY AT COOL TEMPERATURE OR HANG TO DRY FOR LONGER LIFE**
- **OPTIONAL 12 x 8cm IDENTIFICATION LABEL SEWN ON ONE CORNER** simply write on to reduce lost slide sheets
- **S.W.L. 360kg** (Australian Wool Testing Authority)
- **TGA REGISTERED**

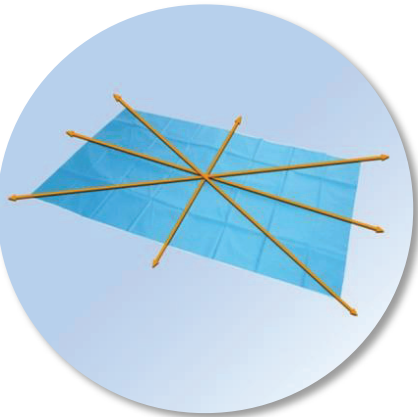

CODE	DESCRIPTION	CTN QTY	SIZE
MT SLIP1	SLIPPERY SALLY Fully Hemmed Reusable Slide Sheet	50	1.0m x 1.45m
MT SLIP2	SLIPPERY SALLY Fully Hemmed Reusable Slide Sheet	50	2.0m x 1.45m
MT SLIP3	SLIPPERY SALLY Fully Hemmed Reusable Slide Sheet	50	3.0m x 1.45m
MT PSB1	SLIPPERY SALLY Heat Sealed Reusable Slide Sheet	50	1.0m x 1.45m
MT PSB2	SLIPPERY SALLY Heat Sealed Reusable Slide Sheet	50	2.0m x 1.45m
MT PSB3	SLIPPERY SALLY Heat Sealed Reusable Slide Sheet	50	3.0m x 1.45m





The Premium Slippery Sally Re-usable range of machine washable Tubes slip extremely well in all directions and is the fabric of choice for some of Australia's leading Manual Handling Trainers.


Slippery Sally® Single Patient Use Transfer Tubes give you:

Single Patient Use SallyTubes® are used for lateral transfers and are ideal for areas of high infection control or where only a lateral transfer is required:


- EMERGENCY SERVICES (PICK UP AND DROP OFF) - STRETCHER TO STRETCHER/BED
- THEATRE - BED TO THEATRE TABLE TO BED
- X-RAY - BED TO X-RAY TABLE TO BED
- PATIENT TRANSPORT
- INTENSIVE CARE UNIT

Features and Benefits:

- REDUCE RISKS ASSOCIATED WITH CROSS-CONTAMINATION
- EACH SHEET SEALED IN PE BAG WITH LABEL
- CAN BE DISPOSED OF AFTER SINGLE PATIENT USE
can last 6 months
- NO LAUNDERING COSTS
- WEIGHT CAPACITY 360kg when used with a draw sheet
- EASY IDENTIFICATION bright yellow
- HEAT-SEALED EDGES WITH SEWN JOIN
- RECYCLABLE IN GENERAL USE

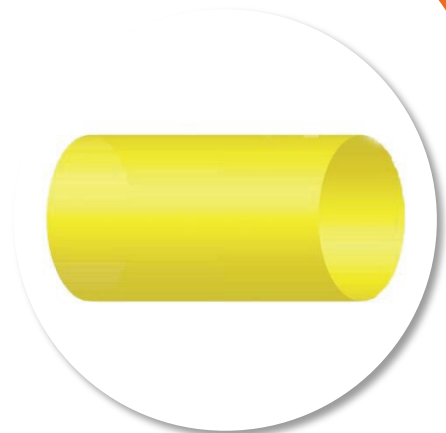

Single Patient Use STRETCHER TO STRETCHER / BED Transfer Tube

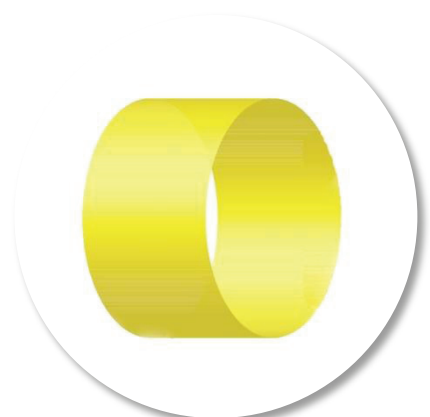

Single Patient Use UP THE BED / STRETCHER Transfer Tube


CODE	DESCRIPTION	CTN QTY	PKT QTY	SIZE
MT STDSS-60	SALLYTUBE Single Patient Use STRETCHER TO STRETCHER / BED Transfer Tube	5	60 per box (5 inner boxes of 12 SallyTubes)	145cm x 72.5cm
MT STDUB-60	SALLYTUBE Single Patient Use Single Patient Use UP THE BED/STRETCHER Transfer Tube	5	60 per box (5 inner boxes of 12 SallyTubes)	72.5cm x 120cm




The Premium Slippery Sally Reusable range of machine washable Tubes slip extremely well in all directions and is the fabric of choice for some of Australia's leading Manual Handling Trainers.

Slippery Sally® Reusable Transfer Tubes:


- SILICON COATED EASY TO USE AND very slippery
- MACHINE WASHABLE UP TO 75° can be tumble dried at a cool temperature or be hung up to dry for a longer life
- SWL OF 360kg AND TGA REGISTERED

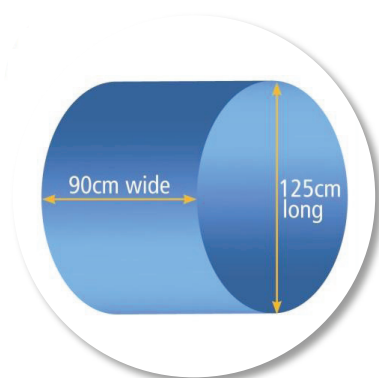

Multi-Directional UP THE BED Transfer Tube

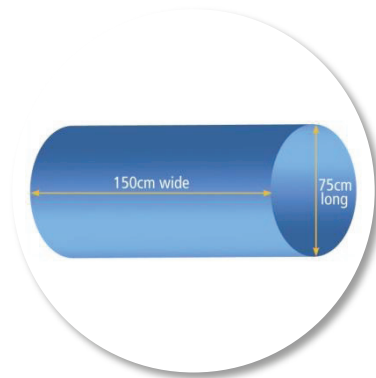

Multi-Directional BED TO BED Transfer Tube

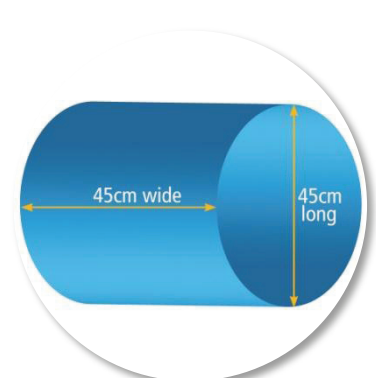

Multi-Directional CHAIR SIZE Transfer Tube

CODE	DESCRIPTION	SIZE
MT UTUBE	SALLYTUBE Reusable Multi-Directional UP THE BED Transfer Tube	90cm (w) x 250cm (circumference)
MT BTBTUBE	SALLYTUBE Reusable Multi-Directional BED TO BED Transfer Tube	150cm (w) x 150 (circumference)
MT CHTUBE	SALLYTUBE Reusable Multi-Directional CHAIR SIZE Transfer Tube	45cm (w) x 90cm (circumference)

PRODUCT SUMMARY

Order Code	Description	Size	Packaging Units/Ctn
MW1029	FLEXISLIDE STANDARD Single Use Patient Transfer Slide Sheet	89cm x 196cm	50
MW1061	FLEXISLIDE EXTRA WIDE Single Use Patient Transfer Slide Sheet	115x196cm	25
MT SBL	SALLYBOARD LARGE Patient Transfer Board	180cm x 58cm x 5mm	1
MT SLIP1-D50	SLIPPERY SALLY Single Patient Use Transfer Slide Sheet	1.0m x 1.45m	50
MT SLIP2-D50	SLIPPERY SALLY Single Patient Use Transfer Slide Sheet	2.0m x 1.45m	50
MT SLIP1	SLIPPERY SALLY FULLY HEMMED Reusable Transfer Slide Sheet	1.0m x 1.45m	50
MT SLIP2	SLIPPERY SALLY FULLY HEMMED Reusable Transfer Slide Sheet	2.0m x 1.45m	50
MT SLIP3	SLIPPERY SALLY FULLY HEMMED Reusable Transfer Slide Sheet	3.0m x 1.45cm	50
MT PSB1	SLIPPERY SALLY HEAT SEALED Reusable Transfer Slide Sheet	1.0m x 1.45m	50
MT PSB2	SLIPPERY SALLY HEAT SEALED Reusable Transfer Slide Sheet	2.0m x 1.45m	50
MT PSB3	SLIPPERY SALLY HEAT SEALED Reusable Transfer Slide Sheet	3.0m x 1.45cm	50
MT STDSS-60	SALLYTUBE Single Patient Use STRETCHER TO STRETCHER / BED Transfer Tube	145cm x 72.5cm	50
MT STDUB-60	SALLYTUBE SALLYTUBE Single Patient Use Single Patient Use UP THE BED/STRETCHER Transfer Tube	72.5cm x 120cm	50
MT UTUBE	SALLYTUBE Reusable Multi-Directional UP THE BED Transfer Tube	90cm (w) x 250cm (circumference)	1
MT BTBTUBE	SALLYTUBE Reusable Multi-Directional BED TO BED Transfer Tube	150cm (w) x 150 (circumference)	1
MT CHTUBE	SALLYTUBE Reusable Multi-Directional CHAIR SIZE Transfer Tube	45cm (w) x 90cm (circumference)	1





PATIENT TRANSFER RANGE

AUSTRALIA

customerservice@globalmedicsgroup.com

www.globalmedicsgroup.com

+1300 197 261

NEW ZEALAND

customerservice@globalmedics.co.nz

www.globalmedics.co.nz

0800 GLOMED 45 66 33



GlobalMedics
Medical and Hygiene Supply Group