



PATIENT TRANSFER RANGE



THE COMPANY

Global Medics is a family owned company originally established in the 1990's to provide quality product and prompt, professional service to the healthcare market. Ongoing contact with a wide spectrum of medical providers has seen our range expand to include major international brands and our own branded products in four key areas - Aged Care, Theatre/Surgical, Pharmacy/IT and Hospital.

PARTNERS FOR LIFE

Throughout our history, we have strived to forge ongoing relationships with both our customers and our suppliers, allowing us to better meet your needs and provide the best, most efficient and cost effective solutions to improving medical care for your residents, patients and your own healthcare teams. Truly understanding what you do and working together in this way, we can help enhance quality of life for all patients.



GLOBAL SOLUTIONS Today our range includes a comprehensive supply of medical and healthcare requirements from gloves to wound care, infection control, theatre and surgical items and more that really deliver results. To view our full product range please visit www.globalmedics.co.nz

OUR MISSION

To forge partners for life so we are better able to deliver the highest quality, innovative products, tested and proven in the global healthcare arena

To reflect the value we place on these partnerships by delivering unbeatable service and backup support

To deliver the most simple and effective solutions that focus on improving quality of life

FOR ALL YOUR MEDICAL PRODUCT ENQUIRIES, BROCHURES OR ORDERS PLEASE CONTACT OUR HELPLINE

0800 GLOMED 45 66 33

Ph 09 920 9900

Fax 09 920 9901

customerservice@globalmedics.co.nz

www.globalmedics.co.nz



Global Medics
Medical and Hygiene Supply Group



INDEX

Global Medics seeks continuously to deliver optimum solutions to your requirements through a range of unique, internationally branded medical products sourced from round the globe.

This brochure outlines our range of Patient Transfer products.

GLOBAL MEDICS Overview	1
Patient Transfer Contents	2
FLEXISLIDE Single Use Patient Transfer Slide Sheet	3-4
SALLYBOARD Patient Transfer Board	5
SLIPPERY SALLY Single Patient Use Slide Sheet	6
SLIPPERY SALLY Reusable Fully Hemmed Slide Sheet	7
SLIPPERY SALLY Single Patient Use Transfer Tubes	8
SLIPPERY SALLY Reusable Transfer Tubes	9
ORDER FORM	10




www.globalmedics.co.nz

SINGLE USE PATIENT TRANSFER SLIDE SHEET

FLEXISLIDE™ is a single use, lateral transfer slide sheet, designed to make patient handling easy and safe, by helping to prevent back strain and injury and assisting in the prevention of cross infection. It is ideal for use in departments across the hospital when moving a patient between two surfaces in a pre/peri-operative OR ER environment.

The product is lightweight and inexpensive but high quality and immensely strong - the material has been tested to over 400kg, wet or dry.

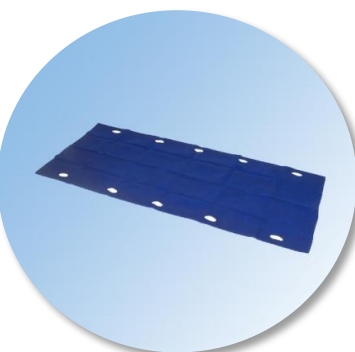

- **COMPACT AND EASY TO STORE**
- **'EASY-SLIDE' DESIGN** - sufficiently stable to remain stationary on a tilted operating table, but slides easily to move the patient in a controlled manner
- **LATEX FREE**
- **WATER RESISTANT**
- **REMAINS UNDER THE PATIENT** so is always ready for use
- **'BLUE COLOUR'** ensures quick, easy identification
- **REPLACES UNNECESSARY SHEETS** etc thereby reducing laundry costs and wastage
- **BARRIER AGAINST BACTERIA**
- **DEDICATED HAND HOLES** ensuring secure grip and reduced wrist stress (refer below)

ADVANTAGES OF DEDICATED HAND HOLE FOR GRIPPING

FLEXISLIDE™ also features ten dedicated hand holes for gripping that minimise awkward twisting of the wrist and ensure a secure hand hold. These handles reduce the risk of injury and wrist stress by enabling the user to hold the slide with upturned palms, using the stronger muscles of the arms. (Gripping with the palms down uses weaker muscles in the arms and results in elevation of the shoulder girdle. This puts the shoulder, elbow, and wrists joints in awkward postures under load, which increases the risk of injury, e.g. refer to picture on left).



INSTRUCTIONS FOR USE

Always risk-assess each patient manoeuvre following the guidelines in your hospital 'Movement and Handling Policy'. Take in to account factors such as patient size and weight. Always ensure that the correct numbers of staff are available before making any transfer.



1. Ensure that all trolley/bed brakes are engaged.
2. Always use a bridging device between the 2 surfaces when transferring patients (e.g. Pat Slide or Easy slide).
3. Place Flexislide™ on the trolley, ensuring that the shiny / hemmed side is facing the trolley surface.
4. Always place the patient's pillow on top of the Flexislide™ and under the patient's head.
5. Ensure the Flexislide™ is as flat as possible underneath the patient's head and feet.
6. To transfer the patient use the hand hole most comfortable for your own height and size.
7. When transferring patients always use a combined push and pull technique as recommended in your hospital moving and handling policy.
8. Flexislide is capable of withstanding the lateral transfer of patients weighing up to 424KG when using a combined push and pull technique
9. To remove or place a new Flexislide™ under an immobile patient do so as if changing a bed sheet. Flexislide™ is folded to assist you in this process.
10. Once the patient has been returned to their bed remove the Flexislide™



IMPORTANT NOTES

- **NEVER LIFT** a patient using Flexislide™
- When transferring patients **DO NOT** use extension handles or straps of any type.
- It's important to ensure that fluids are not left to pool on the surface of the Flexislide™ or beneath the patient (use of absorbent pads may help here).
- If the Flexislide™ has remained clean it can be kept with the patient and can continue to be used for additional lateral transfers while that specific patient remains in hospital.



CODE	DESCRIPTION	CTN QTY	SIZE
MWI061	FLEXSLIDE Extra Wide Transfer Slide Sheet	25	115x196cm
MWI029	FLEXISLIDE Standard	50	89x196cm




WHY USE THE SALLYBOARD RANGE OF PATIENT TRANSFER BOARDS?

WITH 4 GRIP STRIPS ON UNDERSIDE

LIGHTWEIGHT FOR EASY PATIENT TRANSFER


- **CONSTRUCTED FROM LIGHTWEIGHT, HDPE** – much lighter than many transfer boards on the market
- **HANGING HOLE UP ONE END**
- **S.W.L. 300KG (NATA APPROVED LABORATORY)** for Medium, Large and Theatre SallyBoards
- **TGA REGISTERED**
- **MEDIUM, LARGE AND THEATRE SALLYBOARDS** all have 3 handles on each side for ease of manoeuvring
- **SMOOTH TOP SURFACE AND CURVED EDGES FOR EASY PATIENT TRANSFER**
- **4 GRIP STRIPS ON UNDERSIDE TO KEEP BOARD IN PLACE**
- **GOOD LEVEL OF FLEXIBILITY IN THE BOARD** where bed / barouche heights are not always at the same level
- **RADIOLUCENT** – can be used for X-rays

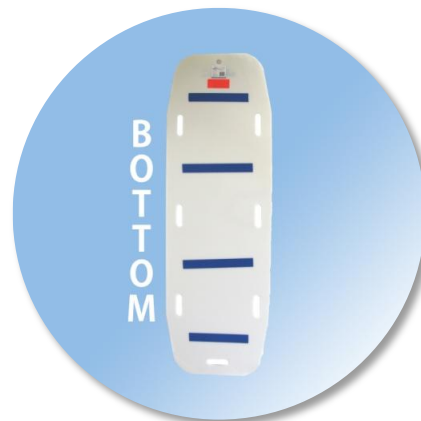

CODE	DESCRIPTION	CTN QTY	SIZE
MT SBL	SALLYBOARD LARGE Patient Transfer Board	1	180cm x 58cm x 5mm



SLIPPERY SALLY® SINGLE PATIENT USE SLIDE SHEETS can greatly improve infection control as well as improve manual handling over reusable slide sheets. During general ward use, Slippery Sally Single Patient Use Slide Sheets can be used for several months and simply wiped down as needed or, in high infections, may be disposed of after use.

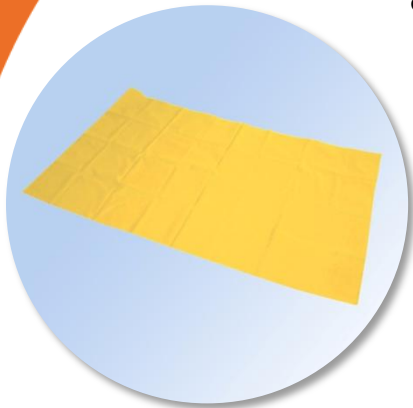

SLIPPERY SALLY® SINGLE PATIENT USE SLIDE SHEETS GIVE YOU:

- REDUCED SPREAD OF INFECTION WHICH MAY OCCUR THROUGH REUSE OF SLIDE SHEETS
- SIGNIFICANT COST SAVINGS
- NO LAUNDERING COST
- ABILITY TO QUICKLY GRAB ONE IN AN EMERGENCY AND USE ON EITHER SIDE
 - very slippery on both sides
- REDUCED CHANCE OF INFECTION - 190T polyester taffeta - S.W.L. 360kg
- EASY IDENTIFICATION - bright yellow
- IMPROVED MANUAL HANDLING - no excuse for staff not to use a slide sheet because they cannot find it or it has not returned from the laundry



SLIPPERY SALLY® SINGLE PATIENT USE SLIDE SHEETS ARE USED BY:

- THEATRE / EMERGENCY DEPARTMENTS / AMBULANCE SERVICES / MRSA PATIENTS
- HOME CARE SERVICES - Ideal for short term clients
- HOSPITALS / AGED CARE WHERE EXPENSIVE SLIDE SHEETS ARE LOST THROUGH THEFT OR LINEN SERVICING

CODE	DESCRIPTION	CTN QTY	SIZE
MT SLIPI-D50	SLIPPERY SALLY Single Patient Use Slide Sheet	50	1.0m x 1.45m
MT SLIP2-D50	SLIPPERY SALLY Single Patient Use Slide Sheet	50	2.0m x 1.45m



The PREMIUM SLIPPERY SALLY® REUSABLE range of machine washable slide sheets has a slip coating on both sides and a quiet soft feel. This Premium Slippery Sally® slips extremely well and consistently on both sides and in all directions. It is the fabric of choice for some of Australia's leading manual handlers.




PREMIUM SLIPPERY SALLY® REUSABLE SLIDE SHEETS ARE AVAILABLE IN TWO OPTIONS:

- **HEAT SEALED EDGES** - reduces the chance of infection whereas a hem can store pathogens.
- **FULLY-HEMMED** - with a 12 x 8cm identification label sewn on is great for reducing Slippery Sally loss at the laundry as well as allocating Slippery Sally's by ward or patient.

PREMIUM SLIPPERY SALLY® REUSABLE SLIDE SHEETS:

- **SILICON COATED BOTH SIDES** - easy to use and very slippery
- **SLIPS CONSISTENTLY IN ALL DIRECTIONS / BOTH SIDES** – no right way up
- **MADE OF SOFT POLYESTER TAFFETA**
- **RETAIN THEIR SLIP FOR OVER 150 WASHES**
- **MACHINE WASHABLE UP TO 75 DEGREES**
- **TUMBLE DRY AT COOL TEMPERATURE OR HANG TO DRY FOR LONGER LIFE**
- **OPTIONAL 12 X 8CM IDENTIFICATION LABEL SEWN ON ONE CORNER**
 - simply write on to reduce lost slide sheets
- **S.W.L. 360kg** (Australian Wool Testing Authority)
- **TGA REGISTERED**

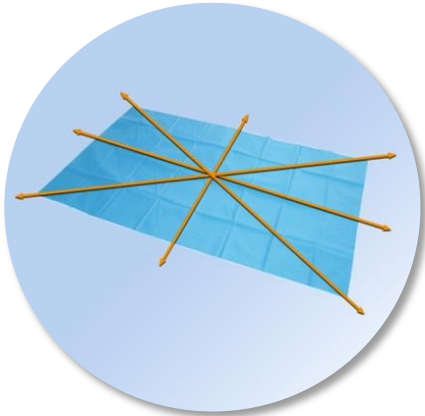

CODE	DESCRIPTION	CTN QTY	SIZE
MT SLIP1	SLIPPERY SALLY Fully Hemmed Reusable Slide Sheet	50	1.0m x 1.45m
MT SLIP2	SLIPPERY SALLY Fully Hemmed Reusable Slide Sheet	50	2.0m x 1.45m
MT SLIP3	SLIPPERY SALLY Fully Hemmed Reusable Slide Sheet	50	3.0m x 1.45m
MT PSB1	SLIPPERY SALLY Heat Sealed Reusable Slide Sheet	50	1.0m x 1.45m
MT PSB2	SLIPPERY SALLY Heat Sealed Reusable Slide Sheet	50	2.0m x 1.45m
MT PSB3	SLIPPERY SALLY Heat Sealed Reusable Slide Sheet	50	3.0m x 1.45m



The Premium Slippery Sally Reusable range of machine washable Tubes slip extremely well in all directions and is the fabric of choice for some of Australia's leading Manual Handling Trainers.


SLIPPERY SALLY® SINGLE PATIENT USE TRANSFER TUBES GIVE YOU:

Single Patient Use SallyTubes® are used for lateral transfers and are ideal for areas of high infection control or where only a lateral transfer is required:

- EMERGENCY SERVICES (PICK UP AND DROP OFF) - STRETCHER TO STRETCHER/BED
- THEATRE - BED TO THEATRE TABLE TO BED
- X-RAY - BED TO X-RAY TABLE TO BED
- PATIENT TRANSPORT
- INTENSIVE CARE UNIT



Single Patient Use STRETCHER TO STRETCHER / BED Transfer Tube

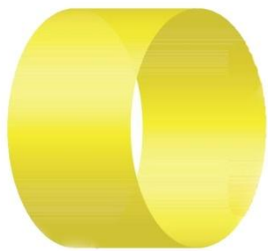

Single Patient Use UP THE BED / STRETCHER Transfer Tube

FEATURES AND BENEFITS:

- REDUCE RISKS ASSOCIATED WITH CROSS- contamination
- EACH SHEET SEALED IN PE BAG WITH LABEL
- CAN BE DISPOSED OF AFTER SINGLE PATIENT USE - can last 6 months
- NO LAUNDERING COSTS
- WEIGHT CAPACITY: 360kg when used with a draw sheet
- EASY IDENTIFICATION - bright yellow
- HEAT-SEALED EDGES WITH SEWN JOIN
- RECYCLABLE IN GENERAL USE

CODE	DESCRIPTION	CTN QTY	PKT QTY	SIZE
MT STDSS-60	SALLYTUBE Single Patient Use STRETCHER TO STRETCHER / BED Transfer Tube	5	60 per box (5 inner boxes of 12 SallyTubes)	145cm x 72.5cm
MT STDUB-60	SALLYTUBE Single Patient Use Single Patient Use UP THE BED/STRETCHER Transfer Tube	5	60 per box (5 inner boxes of 12 SallyTubes)	72.5cm x 120cm




The Premium Slippery Sally Reusable range of machine washable Tubes slip extremely well in all directions and is the fabric of choice for some of Australia's leading Manual Handling Trainers.

SLIPPERY SALLY® REUSABLE TRANSFER TUBES:


- BEING SILICON COATED IT MAKES THEM EASY TO USE AND VERY SLIPPERY.
- MACHINE WASHABLE IN UP TO 75 DEGREES OF HOT WATER, AND CAN BE TUMBLE DRIED AT A COOL TEMPERATURE OR BE HUNG UP TO DRY FOR A LONGER LIFE.
- SWL OF 360KG AND TGA REGISTERED.



Multi-Directional UP THE BED
Transfer Tube

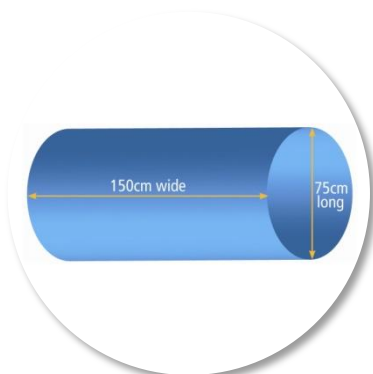

Multi-Directional BED TO BED
Transfer Tube

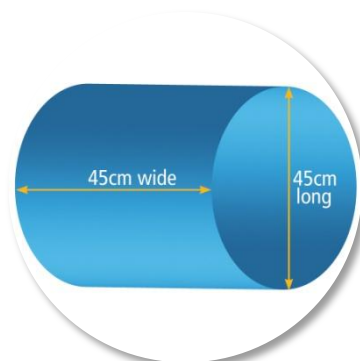

Multi-Directional CHAIR SIZE
Transfer Tube

CODE	DESCRIPTION	SIZE
MT UTUBE	SALLYTUBE Reusable Multi-Directional UP THE BED Transfer Tube	90cm (w) x 250cm (circumference)
MT BTBTUBE	SALLYTUBE Reusable Multi-Directional BED TO BED Transfer Tube	150cm (w) x 150 (circumference)
MT CHTUBE	SALLYTUBE Reusable Multi-Directional CHAIR SIZE Transfer Tube	45cm (w) x 90cm (circumference)



ORDER FORM

TO ORDER ANY REQUIRED PRODUCT

COMPANY _____

ADDRESS _____

ORDER NO _____ EMAIL _____

ORDERED BY _____ ACCOUNT NO _____ DATE _____

Global Medics Limited 42 Andrew Baxter Drive, Airport Oaks, Auckland 2022. PO Box 97377, Manukau, Auckland 2241
Tel: + 64 9 920 9900 Fax: + 64 9 920 9901 www.globalmedics.co.nz NZ FREEPHONE 0800 GLOMED 45 66 33
Email customerservice@globalmedics.co.nz

REF CODE	DESCRIPTION	SIZE	NO. PER CTN	CTNS ORDERED
MW1029	FLEXISLIDE STANDARD Single Use Patient Transfer Slide Sheet	89cm x 196cm	50	
MW1061	FLEXISLIDE EXTRA WIDE Single Use Patient Transfer Slide Sheet	115x196cm	25	
MT SBL	SALLYBOARD LARGE Patient Transfer Board	180cm x 58cm x 5mm	1	
MT SLIPI-D50	SLIPPERY SALLY Single Patient Use Transfer Slide Sheet	1.0m x 1.45m	50	
MT SLIP2-D50	SLIPPERY SALLY Single Patient Use Transfer Slide Sheet	2.0m x 1.45m	50	
MT SLIPI	SLIPPERY SALLY FULLY HEMMED Reusable Transfer Slide Sheet	1.0m x 1.45m	50	
MT SLIP2	SLIPPERY SALLY FULLY HEMMED Reusable Transfer Slide Sheet	2.0m x 1.45m	50	
MT SLIP3	SLIPPERY SALLY FULLY HEMMED Reusable Transfer Slide Sheet	3.0m x 1.45cm	50	
MT PSBI	SLIPPERY SALLY HEAT SEALED Reusable Transfer Slide Sheet	1.0m x 1.45m	50	
MT PSB2	SLIPPERY SALLY HEAT SEALED Reusable Transfer Slide Sheet	2.0m x 1.45m	50	
MT PSB3	SLIPPERY SALLY HEAT SEALED Reusable Transfer Slide Sheet	3.0m x 1.45cm	50	
MT STDSS-60	SALLYTUBE Single Patient Use STRETCHER TO STRETCHER / BED Transfer Tube	145cm x 72.5cm	50	
MT STDUB-60	SALLYTUBE SALLYTUBE Single Patient Use Single Patient Use UP THE BED/STRETCHER Transfer Tube	72.5cm x 120cm	50	
MT UTUBE	SALLYTUBE Reusable Multi-Directional UP THE BED Transfer Tube	90cm (w) x 250cm (circumference)	1	
MT BTBTUBE	SALLYTUBE Reusable Multi-Directional BED TO BED Transfer Tube	150cm (w) x 150 (circumference)	1	
MT CHTUBE	SALLYTUBE Reusable Multi-Directional CHAIR SIZE Transfer Tube	45cm (w) x 90cm (circumference)	1	



PATIENT TRANSFER RANGE

FREEPHONE 0800 GLOMED 45 66 33
PHONE 09 920 9900 | FAX 09 920 9901
EMAIL customerservice@globalmedics.co.nz
WEB www.globalmedics.co.nz



Global Medics
Medical and Hygiene Supply Group