

clinell®



CLINELL WARMER

for Bed Bathing Products

Clinell Warmer is suitable for use with Clinell Chlorhexidine Wash Cloths, Clinell Chlorhexidine Shampoo cap, Clinell Bedbath Wipes, Clinell Shampoo Cap and Clinell Continence Care Wipes. Once activated, the Clinell Warmer will automatically heat up to the correct temperature. The wipes can be kept inside the warmer for a maximum of 30 days.

Introduction:

1. Ensure the Clinell Warmer is installed upright on a stable, flat surface and plugged in to a mains power socket.
2. Place Clinell Chlorhexidine Wash Cloths, Clinell Bed Bath or Clinell Continence Care packets in between the slots of the racks, then close the door.
3. Switch on by pressing the power button (the red light will be illuminated).
4. Press the 'Warm' button to activate the Warmer. The Warmer will automatically maintain the correct temperature.
5. The sterilize button can be used to provide some antibacterial cleaning of the inside of the warmer and the outside of the wipe packets. If this is required, press the sterilize button and leave on for at least 30 minutes.
6. Before cleaning always disconnect the power plug and let the unit cool down for at least 2 hours. Once cooled remove the 2 racks and clean these with a Clinell Universal Sanitising or Clinell Detergent Wipe, then clean the inside of the warmer with a Clinell Universal Sanitising or Clinell Detergent Wipe. Return the trays and leave to dry for at least 2 hours before switching the unit back on.

Attention:

- The top and bottom of the inside of warmer are hot, so avoid touching them.
- Do not use any other products inside the warmer.
- Do not lie the wipe packets flat or stack them on top of each other.
- Do not keep the wipes in the warmer for more than 14 days.
- Avoid unnecessarily opening of the door when the warmer is in the process of heating up.
- Before using the wipes on patient check the temperature to ensure it is not too hot.
- If you experience any problems with this product, please contact GAMA Healthcare.

Main Techniques Index:

Model	WTK-16L CWARM
Volume	30L
Wattage	150W
Voltage	220V
Frequency	50Hz/60Hz
Temperature	45 ± 10°C
Loading Capacity	20 – 40 Wipes
Dimension	455×385×395mm